Dolphin & Barracuda wheelchair

Item no. 271340 - 271357, 271440 - 271457, 271840 - 271851 & 272044 + 272048

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1. Introduction

Thank you for buying a high-quality Mobilex product. These user instructions contain a detailed product description and important guidelines for correct use of the product. You must read the instructions carefully before using the product. Please always observe all guidelines, in particular the security instructions, in order to use the product safely for a long time. The wheelchairs "Dolphin" and "Barracuda" have been designed for persons with limited motion abilities, it enables them transportation and moving, can be used indoor as well as outdoor. Users can move with the wheelchair independently or with a help of attendant. The wheelchair has been tested and fulfills all requirements imposed by the law. Materials used in the wheelchair fulfill strength requirements. Upon a correct usage of the wheelchair we foresee its failure-free work for years.

Indications

Manual wheelchairs are indicated for people with mobility limitations who have problems with walking, standing or sitting without stable support. The wheelchair is indicated for use to move around in a sitting position, independently or with the help of an attendant. Using the wheelchair for purposes other than those indicated above, is prohibited.

Contraindications

The wheelchair should not be used by people who cannot maintain a stable sitting position. Contraindications cannot be related to one specific clinical picture and they depend on individual evaluation of potential user's health status. The choice of orthopedic aid appropriate for a given clinical picture must be always made with the help of a specialist physician or physiotherapist. In addition, observe the safety information in chapter 2.

If you have any questions, please contact your dealer or Mobilex A/S.

Mobilex A/S
Grønlandsvej 5
DK-8660 Skanderborg
Denmark

Phone: +45 87 93 22 20
Telefax: +45 87 93 17 77
Email: info@mobilex.dk

2. Safety requirements

- Only use the product in flawless condition.
- If defects or errors are detected you must immediately contact the dealer.
- Be aware of the information on the product label.
- Only use the product for the described purpose.
- Don't climb on to the device
- The device must only be used on stable surface.
- When adjusting the device the user must take care not to squeeze the fingers or other body parts.


3. Description

Wheelchair provides:

- A comfortable transportation and storage thanks to its crossed frame structure and wheels with a quick-release axle
- Easiness of maneuvering
- Easy transfer to - and out of the wheelchair
- The durability and the solidity of the structure

The wheelchairs consists of the following elements:

1. Push handles -height-adjustable, enabling the attendant maneuvering the wheelchair.
2. Backrest - made of smooth incombustible material, enabling also easy upholstery cleaning. On Barracuda the angle is adjustable into 3 different positions.
3. Armrest - adjustable in height. It is used as a comfortable support of forearms. It also facilitates getting in and out of the wheelchair.
4. Main wheels - Majority of the user's weight rests on these 24 inches pneumatic wheels. Wheels can be easily taken off, which facilitates the transportation of the wheelchair.
5. Push rim - Thanks to push rims fastened to main wheels self propelled drive of the wheelchair is possible.
WARNING! Do not put fingers between the wheel and the handrims. It threatens the injury of the palm.
6. Front wheels - 8 inches turning wheels are installed on rotating forks, which makes maneuvering of the wheelchair possible.
7. Cross -thanks to it the wheelchair can be easily fold for transportation and storage.

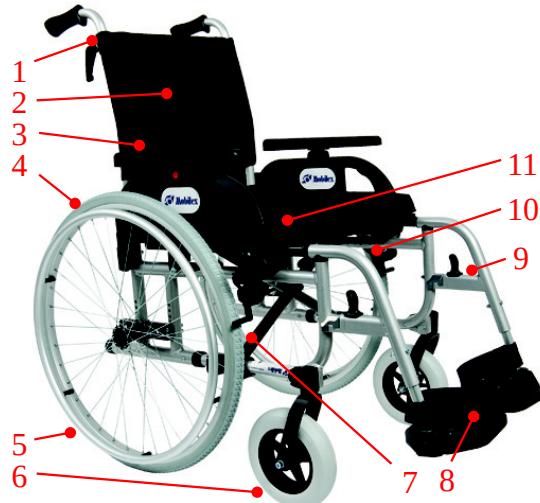8. Footplate - User's feet rest on it. It is possible to lift up the footrest, which can ease getting in and out of the wheelchair.
9. Footrest - The removable footrest is a safe and comfortable support for legs. The footrest is adjustable in length.
10. Brake - Pushing of brake lever towards front causes their pulling and protects the wheelchair against unforeseen movement and the rolling down the slope.
11. Seat - Made in similar way to the backrest.

WARNING! Always pull both brakes, if you do not move on the wheelchair.

Function description

Footrests

Standard footrests can be disassembled or pushed away on side.

To incline and disassemble the footrest it has to be unblock by pulling out the safety-lock (1) and to turn the footrest outside, then pull it upwards (2). The footrest assembling should be conducted in reverse order by inserting it in base of frame and turning it towards. The safety-lock will block self-actively.

The wheelchair is equipped with height-adjustable footrests. To adjust their length one should slacken a screw (3), choose the height of the footrest and tight the screw. During using the wheelchair outside, the footrests must be placed on height of minimum 4-5cm from the base. In addition, the footrests posses the ability of moving the footplates upwards (4), which facilitates getting in and out of the wheelchair and also disassembling the footrests.

WARNING! You should never climb the footrests, it can cause falling over the wheelchair. You should never raise the wheelchair using the footrests.

Armrests

The armrests are height adjustable. To change the position one press the red button (1), choose the height of the armrest and then release the button.

The armrests can be also inclined upwards, which ease getting in and out of the chair.

To unblock the safety-lock (2) one should press the lever (2) placed on the side of the armrest and then lift the side panel up.

After lowering the armrest and placing it on the guide bar, the safety-lock (2) block self actively.

Brakes

The wheelchair is equipped with parking brakes on both rear wheels to prevent rolling away.

To apply the brake push the lever forward (1). To release the brake pull the lever toward the wheel (2).

Always apply the brakes on both sides of the wheelchair.

Warning!

The parking brakes are not designed as driving brakes and only serve to lock the wheelchair (park position). They may not be used for braking while driving!

24" rear wheels

The wheelchair is delivered with the standard aluminum push rims. The material and the distance between the push rim and the wheel can influence on the grip strength of the wheelchair's user.

One should be careful and not allow the fingers to get between spokes during driving in narrow passages. To avoid such risk it is recommended to use a spoke protection.

One should regularly control the pressure in tyres, which ought to be in range of 2 - 2,2 atm depending on the weight of the user.

The wheels on "Dolphin" and "Barracuda" chair can additionally be easily removed and fitted in by pressing the quick-release button placed in the center of the wheel (1).

The wheel balance on the "Dolphin" and "Barracuda" can be additionally adjusted by moving the wheel axle. In his case it is necessary to adjust also the position of the turning wheels.

WARNING! We recommend such adjustment to be done by the qualified staff. An inappropriate adjustment can cause deterioration of the safety usage of the wheelchair and decrease its durability.

Safety belts

There is possibility to attach the hip safety belts. They are not among the standard equipment.

Push handles

The push handles are height-adjustable

To adjust the height of the push handles must release the lever (1) by turning it counter-clockwise. Then adjust the handlebar to the desired height (2) and screw the lever (1) again by turning it clockwise.

WARNING! To ensure a secure push both handles must be set to the same height.

WARNING! The push handles has to be used only for pushing and steering the wheelchair. You should NOT use it for lifting or tilting the wheelchair!


Backrest with angle adjustment (only on Barracuda)

To adjust the backrest angle


- Press and hold the red button (1)
- Use the second hand to adjust the backrest to desired angle (2)
- Release button (1) to secure backrest in desired position
- Repeat above steps on the other side of the backrest

WARNING! Always ensure that the angle of both back tubes is adjusted to the same positions.

How to fold and unfold the wheelchair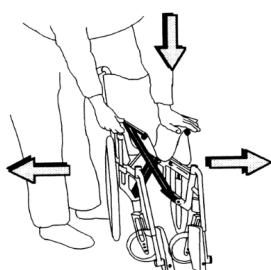

Placing one hand on the seat upholstery press one of the tubes, which support the seat upholstery (in accordance with the instructions on the drawing). One should be careful and not put the hand between tubes and the wheelchair frame during unfolding it.

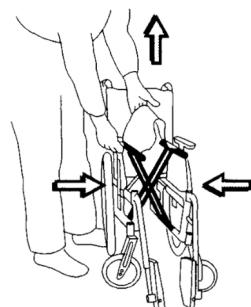Tip pedals

The tip pedal on wheelchairs makes it easier for the attendant to avoid obstacles by tilting the wheelchair backwards (see photo left). The tip pedal is built into the frame (see photo right).

NOTE! DO NOT tilt the wheelchair with the handles, but use the tip-pedal!

Advanced settings and adjustments

Warning! We recommend that all advanced adjustments are only carried out by qualified personnel. An unsuitable adjustment may affect safety when using the wheelchair and reduce the service life of the wheelchair.

Adjusting the seat depth

With the Mobilex Barracuda and Dolphin wheelchairs, the seat depth can be easily adjusted to suit the size of the user.


WARNING! The correct seat depth should always be professionally determined by a therapist or specialist!

To adjust the seat depth, proceed as follows:

1. Loosen and remove the screws (1) on the upper and lower frame tubes.
- 2 Now slide the inner frame tube (2) forwards or backwards until you have reached the correct setting.
3. Replace the screws (1) and tighten them.
4. Repeat the procedure on the opposite side of the wheelchair.
5. After adjusting the seat depth it may be necessary to adjust the position of the brakes.
6. After adjusting the seat depth, the seat cushion must also be adjusted to the new seat depth. To do this, open the cover, cut off the foam at the specified notches and close the cover again using Velcro.

Adjusting the seat height / height adjustment of the rear wheels

If necessary, the seat height of the wheelchair can be adjusted by raising or lowering the axles of the rear wheels. In this case, the position of the front wheels must also be adjusted.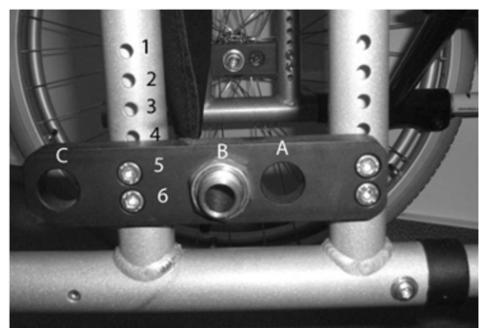

It is possible to adjust the rear wheels up or down, forwards or backwards to suit the tyre size and the personal preferences of the user. The height adjustment of the wheel axles allows the adjustment of the seat height. The horizontal alignment (front / back) allows to influence the stability and manoeuvrability of the chair. The most common and neutral position is "B". If it is necessary to increase the manoeuvrability of the wheelchair, position "A" can be used. With the wheels in position "A" it is easier to manoeuvre the wheelchair but the stability to the rear is significantly reduced. If the wheels are set to position "A" the chair should always be equipped with an activated anti-tip. Persons with amputated leg(s) should never use the wheelchair with the wheels in position "A" - the risk of tipping is too high. Leg amputees should set position "C" as it provides greater stability than the other positions. Even when using this setting, the wheelchair should be equipped with anti-tippers (accessories). As the manoeuvrability of the wheelchair is made more difficult with the axles in position "C", this setting is only recommended for leg amputees.

WARNING! The correct position of the wheels should only be determined by professional therapists or specialist companies.


There are 5 different height settings and 3 different horizontal settings possible.

To adjust the axle height, proceed as follows:


1. remove the wheels with the quick-release axle.
2. remove the black axle mounting plate by loosening the 4 screws. Remove the black axle mounting plate by loosening the 4 screws.
3. place the black axle mounting plate in one of the 5 positions (holes 1 and 2, 2 and 3). (hole 1 and 2, 2 and 3, 3 and 4, 4 and 5 or 5 and 6).
4. Fix the axle mounting plate again with the 4 screws.
5. Repeat the process with the opposite axle mounting plate.

To adjust the axles horizontally (A, B or C) proceed as follows:

1. remove the wheels with the quick-release axle.
2. unscrew the silver-coloured bush from the axle mounting plate
3. put the beech back in the desired position (A, B or C). 4. screw it tight.
4. screw it tight
5. repeat the procedure with the opposite bush.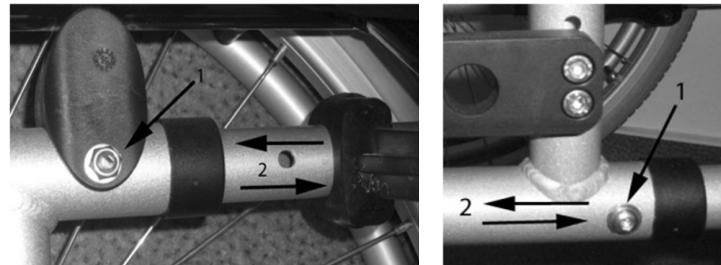

8" front wheels

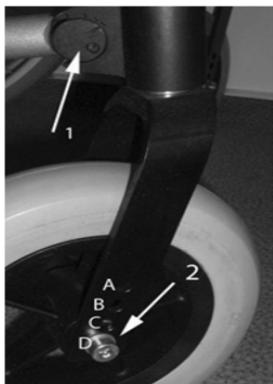The wheelchair is equipped with PU wheels 200x50 mm as standard. We recommend to stay with this type of tyres, but it is also possible to use pneumatic tyres. It is possible to mount different sizes of front wheels. This is made necessary by adjusting the rear axles (see also the table below). If the vertical position of the rear wheels is changed, it is necessary to adjust the position of the front wheels accordingly. The vertical axis of rotation of the fork should always be set at a 90° angle to the ground (+/- 2°). It is possible to adjust the desired angle by turning the eccentric screw (1).

If the adjustment range needs to be larger, it may be necessary to move the front wheel axle to another hole (A, B, C, D) by loosening the screw (2) and moving the wheel up or down to the desired height and tightening it again. In some cases it may be necessary to replace the standard front wheels (8") with a wheel of a different size (6" or 7"). The table below shows the possible wheel positions. It provides a clear overview of seat heights, rear wheel positioning holes, correct front wheel sizes and positions.


Correct positions and front wheel sizes depending on desired seat height:

Desired seat height	Position of the axle mounting plate (hole numbers)	Front wheel position (hole designation on the fork)	Correct wheel size at the front
45,0 cm	1+2	B	6"
46,3 cm	2+3	B	7"
47,7 cm	3+4	C	7"
50,0 cm	4+5	C	8"
51,3 cm	5+6	D	8"

Wheelchair transportation

Car transport

Place the wheelchair in the car boot. If, for some reason, it is not possible, put the folded wheelchair between the front and the rear seats inside the car ensuring that the wheelchair is in a sure and stable position. If it is not possible the wheelchair should be fasten with the safety belt.

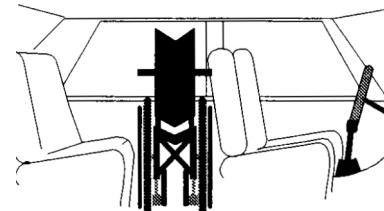WARNING! The wheelchair is not the car seat. During traveling by vehicles seating on the wheelchair is not allowed. The chair should be folded and safely immobilized.

Accessories

As an option, the wheelchairs can be equipped with the following accessories (not part of the standard equipment)

- ✓ Anti-tip
- ✓ Transfer wheels
- ✓ Safety belt
- ✓ Drum brakes
- ✓ Angle and height adjustable footrests
- ✓ Tensionable back (Velcro back)

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4. Technical data

Dolphin:

Dolphin has been successfully crash tested to ISO 7176/19 in its standard version. The Heavy Duty version was not tested. Standard version can be used as a vehicle seat for users weighing 75 kg and 136 kg.

The wheelchair is designed to be forward-facing when used as a seat in a motor vehicle. The occupant should transfer to the vehicle seat and use the vehicle-installed restraint system whenever it is possible! In order to fasten the wheelchair to the floor of the vehicle, only use restraint systems which meet the standard ISO 10542.

Dolphin	Item no. 271340	Item no. 271344	Item no. 271348	Item no. 271351	Item no. 271354	Item no. 271357
Seat-width	40 cm	44 cm	48 cm	51 cm	54 cm	57 cm
Seat-depth adjustable			42 cm, 44 cm, 46 cm, 48 cm			
Seat-height min./max.*			45 - 52,5 cm in steps of 1,5 cm			
Total width	60,30 cm	64,30 cm	68,30 cm	71,30 cm	74,30 cm	77,30 cm
Weight	15,60 kg	15,90 kg	16,20 kg	16,75 kg	19,40 kg	22,64 kg
Max. user weight	150 kg	150 kg	150 kg	150 kg	200 kg without, but only 150 kg with drum brakes	

Dolphin with drum brakes	Item nr. 271440	Item nr. 271444	Item nr. 271448	Item nr. 271451	Item nr. 271454	Item nr. 271457
Seat-width	40 cm	44 cm	48 cm	51 cm	54 cm	57 cm
Seat-depth adjustable			42 cm, 44 cm, 46 cm, 48 cm			
Seat-height min./max.*			45 - 52,5 cm in steps of 1,5 cm			
Total width	60,30 cm	64,30 cm	68,30 cm	71,30 cm	74,30 cm	77,30 cm
Weight	16,10 kg	16,40 kg	16,60 kg	17,25 kg	20,10 kg	23,14 kg
Tyres front/back	PU/PU	PU/PU	PU/PU	PU/PU	PU/air	PU/air
Max. user weight	150 kg	150 kg	150 kg	150 kg	150 kg	150 kg

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
Product label

A product label is placed on the front of the frame.

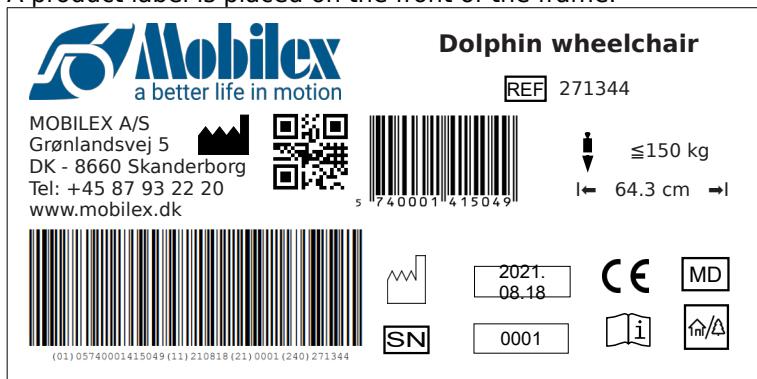Example of product label for Dolphin 271344

	QR-code link to homepage		Manufacturing date
	Manufacturer		Max load
	Observe the user manual		Max width
	To be used indoor and outdoor		serial no
	CE marked product		Item Number
	Max load basket		Medical Device
	Only to be used indoor		

Meaning of the icons

5. Care and maintenance

- ✓ The frame ought to be cleaned with use of damp cloth alternatively with addition of a soft detergent.
- ✓ Caution! The frame should be regularly controlled, to find on time the damages of varnish, which may cause corrosion.
- ✓ The seat and the backrest are made of a very durable strengthened nylon. The seat upholstery can be easily removed by slackening the screws placed on the frame tubes. The backrest upholstery can be removed by slackening the screws placed on the backrest tubes.
- ✓ The seat and the backrest upholstery can be washed with use of sponge and gentle washing soap.

Recommended service life: Normally 10 years depending on use.

Storage: We recommend storage in a dry place.

Reuse

The product is suitable for reuse. Before reusing, where a new user is to take over the product, the product must be checked according to point 5.

Parts, which should be regularly inspected

Part name	Control type	Control frequency
Tyres	Tyre pressure, condition of tread and tyres. One should take care, to keep the pressure in both tyres on the same level. Harder tyres provide better maneuverability of the wheelchair and larger easiness of driving, but they reduce the comfort on uneven surface.	At least once a week
Spokes	Loose spokes can cause deformation of the rear wheels. Contact your point of sale or local bicycle service to manage the problem of loose spokes.	If the problem occurs
Wheel axles	Remove the hair or accumulated dirt	If necessary
Push Rims	Excessively scratched push rims ought to be exchanged because they can wound the user's hands during driving the wheelchair	If necessary
Brakes	The braking force depends on the tyre pressure. The efficiency of brake operation can be also affected with dirt accumulated on tyres. Keep the brakes clean wiping them with a damp cloth to remove dirt, and also lubricating the funnel of screws, on which the brake levers turn.	Control the correctness of brake operation at least once a month
Frame	Maintain the wheelchair in the highest cleanliness for better comfort of the user.	At least once a month
Turning wheels	The area between the fork and the front wheel should be kept clean, because of dirt accumulating there, which can cause faster wear of turning wheel bearings. To do so, one should disassemble the front wheel by unscrewing it from the fork, remove all dirt, and then preserve the metal elements of the wheel (i.e. technical grease).	The maintenance ought to be made once a month or often depending on the soiling degree.
Detachable elements	Check condition of detachable elements of the wheelchair, if screws are loosen they should be tighten.	In case of intensive exploitation of the wheelchair the inspection should be carry out once a month.
Quick-release axles of the rear wheels	One should clean and then lubricate lightly the quick-release axles of the rear wheels, with a damp cloth the slots of fixing the quick-release axles ought to be cleaned up from all dirt. The wheel should be laid on the base, resting on the push rim (never opposite way, because the quick-release axle can be soil, which can cause its seizing in the main axle opening. In case of solid particles occurring on the surface of the quick-release axle of the wheel, using is not allowed.	Lubricating should be carried out periodically.

6. Most common problems and solutions

If you notice any irregularities in the wheelchair working you cannot use it. You should contact the local point of sale or service department of the wheelchair supplier. The manufacturer does not guarantee correctness of the wheelchair operation, if the parts applied in it are not original.

<i>The wheelchair seems to tilt on one side</i>	<ul style="list-style-type: none">• inflate tyres• check fastening of the front turning wheels• check displacement of the wheelchair mass• one of the rear tyres might be inflated more than the other one
<i>The wheelchair is hard to push</i>	<ul style="list-style-type: none">• inflate tyres• check fastening of the front turning wheels• remove dirt or entangled hair from the front turning wheels• to large load applied on the front turning wheels: move the center of gravity
<i>The wheelchair is hard to turn</i>	<ul style="list-style-type: none">• inflate tyres• ensure, that screws fastening the front turning wheels are not tightened to strong• remove dirt or entangled hair from the front turning wheels
<i>Brakes are not working correctly</i>	<ul style="list-style-type: none">• inflate tyres• adjust the distance between the brake and tyres
<i>It is hard to fold and unfold the wheelchair</i>	<ul style="list-style-type: none">• the upholstery fitted to tight• clean up or besmear the cross under the seat
<i>The wheelchair is not stable</i>	<ul style="list-style-type: none">• inflate tyres• ensure, that all screws and nuts are tightened• ensure, the wheelchair is properly adapted to the passenger

For realization of basic service actions the set of socket wrenches, or adjustable spanner, flat and cross screwdriver will be sufficient.

7. Disposal of the product

The product can not be disposed of with household waste but must be brought to the local recycling center.

8. Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

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Mobilex A/S, Grønlandsvej 5, DK-8660 Skanderborg, Denmark, SRN: DK-MF-000021885

Hereby declare under our sole responsibility as a legal manufacturer that the product specified on the product list below, meet the essential health and safety requirements and is in conformance with the provisions of the Regulation (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL on medical devices.

The product specified on the product list below is "technical aid for the disabled", classified as Class I, medical device. The classification is based on the requirements of Rule 1 of annex VIII, of the Regulation (EU) 2017/745.

The CE marking has been affixed on the product according to Annex V of the Regulation (EU) 2017/745.

PRODUCT LIST

Barracuda wheelchair with adjustable back, Aluminium

REF / item no.	271840	271844	271848	271851
UDI-DI	5740001 441031	57400014 41048	5740001 441055	574000144 1062
BASIC-UDI-DI	57400014BARRACUDATB			

Barracuda wheelchair with drum brake, Aluminium

REF / item no.	272044	272048
UDI-DI	5740001441031	5740001441048
BASIC-UDI-DI	57400014BARRACUDATB	

ACCESSORIES LIST

Item no.	Accessories item no.
271840	271082, 271085, 271086, 271075, 271A81, 271A82, 27840H, 271645, 271649
271844	271082, 271085, 271086, 271075, 271A83, 271A84, 27844H, 271646, 271649
271848	271082, 271085, 271086, 271075, 271A85, 271A86, 27848H, 271647, 271650
271851	271082, 271085, 271086, 271075, 271A87, 271A88, 27851H, 271648, 271650
272044	271082, 271085, 271086, 271075, 271A83, 271A84, 27844H, 271646, 271649
272048	271082, 271085, 271086, 271075, 271A85, 271A86, 27848H, 271647, 271650

Harmonized norms used during conformity estimation

PN-EN 12182, PN-EN 12183, ISO 7176-1, PN-ISO 7176-3, PN-ISO 7176-5, PN-ISO 7176-7, PN-ISO 7176-8, PN-ISO 7176-15, PN-ISO 7176-16

EU2017/745 for Medical Equipment
(über Medizinprodukte / for medicinalprodukter / per dispositivi medici)

In case of unintended incidents or suspected faults, failures or defects of the product, the dealer or Mobilex A/S must be contacted /

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Mobilex A/S

Skanderborg, 29-04-2022

A handwritten signature in blue ink, appearing to read 'Thomas N. Ch.'

Signature/Unterschrift/Underskrift/Firma

Declaration of Conformity (CE)

(Konformitätserklärung / Konformitetserklæring / Dichiarazione di conformità/Konformitetsförklaring)

Mobilex A/S, Grønlandsvej 5, DK-8660 Skanderborg, Denmark, SRN: DK-MF-000021885

Hereby declare under our sole responsibility as a legal manufacturer that the product specified on the product list below, meet the essential health and safety requirements and is in conformance with the provisions of the Regulation (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL on medical devices.

The product specified on the product list below is "technical aid for the disabled", classified as Class I, medical device. The classification is based on the requirements of Rule 1 of annex VIII, of the Regulation (EU) 2017/745.

The CE marking has been affixed on the product according to Annex V of the Regulation (EU) 2017/745.

DOC

PRODUCT LIST

Dolphin wheelchair, Aluminium

REF / item no.	271340	271344	271348	271351	271354	271357
UDI-DI	57400014 15032	5740001 415049	574000141 5056	574000141 5063	57400014 15070	57400014 15087
BASIC-UDI-DI						57400014DOLPHINET

Dolphin wheelchair with drum brake, Aluminium

REF / item no.	271440	271444	271448	271451	271454	271457
UDI-DI	57400014 15094	5740001 415100	574000141 5117	574000141 5124	57400014 15131	57400014 15148
BASIC-UDI-DI						57400014DOLPHINET

ACCESSORIES LIST

Item no.	Accessories item no.
271340	271082, 271085, 271086, 271075, 271A81, 271A82, 27840H, 271645, 271649
271344	271082, 271085, 271086, 271075, 271A83, 271A84, 27844H, 271646, 271649
271348	271082, 271085, 271086, 271075, 271A85, 271A86, 27848H, 271647, 271650
271351	271082, 271085, 271086, 271075, 271A87, 271A88, 27851H, 271648, 271650
271354	271082, 271085, 271086, 271075, 27854H
271357	271082, 271085, 271086, 271075, 27857H
271440	271082, 271085, 271086, 271075, 271A81, 271A82, 27840H, 271645, 271649
271444	271082, 271085, 271086, 271075, 271A83, 271A84, 27844H, 271646, 271649
271448	271082, 271085, 271086, 271075, 271A85, 271A86, 27848H, 271647, 271650
271451	271082, 271085, 271086, 271075, 271A87, 271A88, 27851H, 271648, 271650
271454	271082, 271085, 271086, 271075, 27854H
271457	271082, 271085, 271086, 271075, 27857H

Harmonized norms used during conformity estimation:

PN-EN 12182, PN-EN 12183, ISO 7176-1, PN-ISO 7176-3, PN-ISO 7176-5, PN-ISO 7176-7, PN-ISO 7176-8, PN-ISO 7176-15, PN-ISO 7176-16

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Item nr.	Description
271075	Complete headrest with bracket (for wheelchair)
271082	Anti-tip safety wheels with foot pedal - left / right
271085	4" Transfer-wheels Dolphin/Barracuda (1 pair)
271086	Foot pedal for Dolphin/Barracuda
271A81	Height- and angle adjustable leg rest for seat width 40 cm, left
271A82	Height- and angle adjustable leg rest for seat width 40 cm, right
271A83	Height- and angle adjustable leg rest for seat width 44 cm, left
271A84	Height- and angle adjustable leg rest for seat width 44 cm, right
271A85	Height- and angle adjustable leg rest for seat width 48 cm, left
271A86	Height- and angle adjustable leg rest for seat width 48 cm, right
271A87	Height- and angle adjustable leg rest for seat width 51 cm, left
271A88	Height- and angle adjustable leg rest for seat width 51 cm, right
271645	Flip back for wheelchair, width 40 cm
271646	Flip back for wheelchair, width 44 cm
271647	Flip back for wheelchair, width 48 cm
271648	Flip back for wheelchair, width 51 cm
271649	Velcro-back for wheelchair, SW: 40 - 44 cm
271650	Velcro-back for wheelchair, SW: 48 - 51 cm
27840H	Seat cushion for wheelchair, 40 x adjustable 42-48 x 5 cm (W x D x H)
27844H	Seat cushion for wheelchair, 44 x adjustable 42-48 x 5 cm (W x D x H)
27848H	Seat cushion for wheelchair, 48 x adjustable 42-48 x 5 cm (W x D x H)
27851H	Seat cushion for wheelchair, 51 x adjustable 42-48 x 5 cm (W x D x H)
27854H	Seat cushion for wheelchair, 54 x adjustable 42-48 x 5 cm (W x D x H)
27857H	Seat cushion for wheelchair, 57 x adjustable 42-48 x 5 cm (W x D x H)

Assembly instruction

(Montageanleitung / Montagevejledning / Istruzioni per il montaggio/Monteringsanvisningar)

Index	Name	Assembly instruction		
1	Parts delivered			
2	Mounting spoke wheels			
3	Mounting the fork			
4	Installation of wheelchair brake			
5	Fitting the armrest			
6	Installation of leg supports			
6.1	Adjusting the height of footplate + Angle adjustment of footplate			
7	Mounting of push handles			
8	Mounting of the tip pedal			
9	Adjustment of seat depth			
10	Adjusting the cushion			
11	Adjusting seat height			
12	Adjusting the center of gravity of the wheelchair			
13	Adjusting the seat angle			

Assembly instructions

Scan the following QR-code to access our assembly instruction for the Dolphin and Barracuda wheelchairs

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